kinder

kinder

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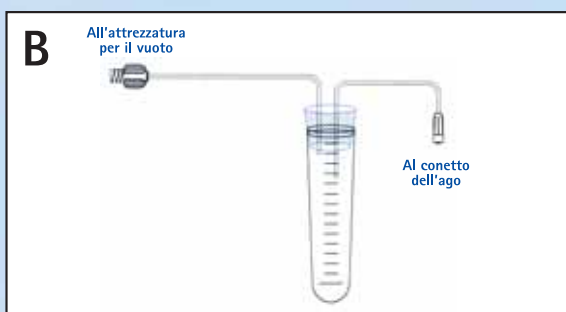
Kinder is a kit for taking oocyte samples intended for "in vitro" insemination by means of FIVET or ICSI technique. The medical device is used to take ultrasound-guided transvaginal samples of the female gamete within a medically assisted procreation (MAP) cycle. This surgical operation requires a needle connected to a set in order to aspirate the follicular fluid containing the oocytes in a closed and sterile system.

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Place the silicone cap on a test tube.

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
Connect the extension with female Luer lock to the vacuum generating equipment.

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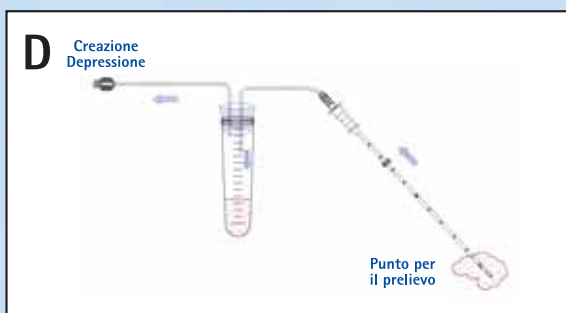Using the ultrasound guide insert the tip of the needle at the point from where the sample is to be taken. The beaked tip is sufficiently long to reach the fornices and penetrate the pelvic cavity for 5-6 cm. The stylet moreover gives the needle rigidity and prevents occlusion of the lumen with fragments of tissue during perforation of the vaginal walls. When the point to be aspirated has been reached, extract the stylet.

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Connect the extension with male Luer lock to the needle cone before taking the sample. Create a vacuum in the test tube and consequently aspirate the follicular fluid.

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